



Thompson Retractor

Uncompromised Exposure™



Thompson RingTrack™ User Manual:
SET UP INSTRUCTIONS FOR THOMPSON RINGTRACK™

RingTrack™ Set Up Instructions

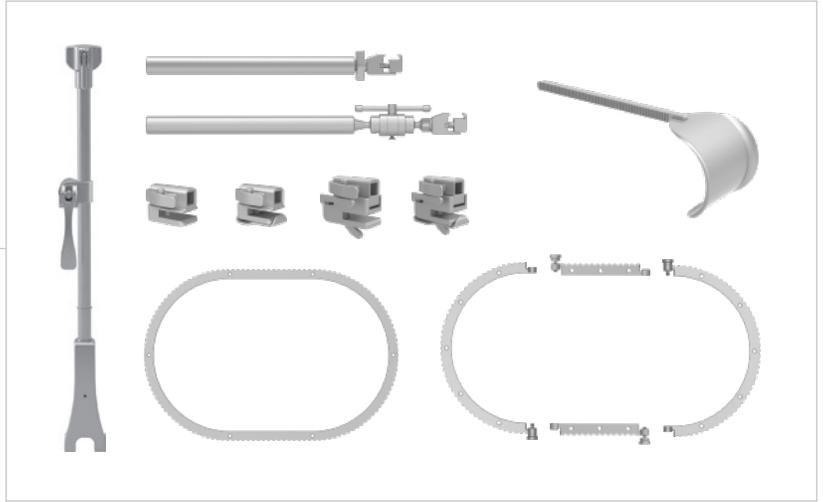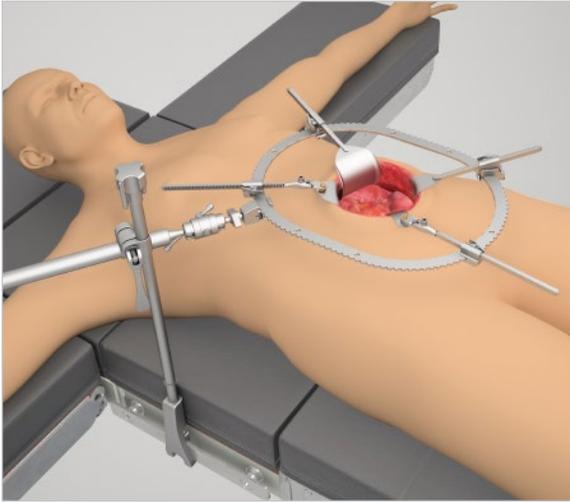

Warnings and Precautions:

-  **CAUTION: Avoid compressing the patient's body with frame components to prevent nerve damage.**
-  **We recommend relaxing tension on retractors every 20 minutes to ensure proper blood flow.**
-  **The Thompson Retractor is provided in a non-sterile condition. Refer to Thompson Retractor IFU for cleaning, sterilization, and care instructions.**

Reference the Thompson Retractor IFU for additional warnings and cautions.

For other Thompson Retractor Systems, reference corresponding User Manual for each.

RingTrack™ Frame Set Up Instructions


STEP 1: RAIL CLAMP

Place Elite Rail Clamp onto the rail over the sterile drape and turn top knob clockwise to tighten.




STEP 2: HORIZONTAL BAR

Insert horizontal bar into joint on the Elite Rail Clamp, position as desired, ensuring ring clamp is horizontal and facing up. Flip cam joint handle to lock.

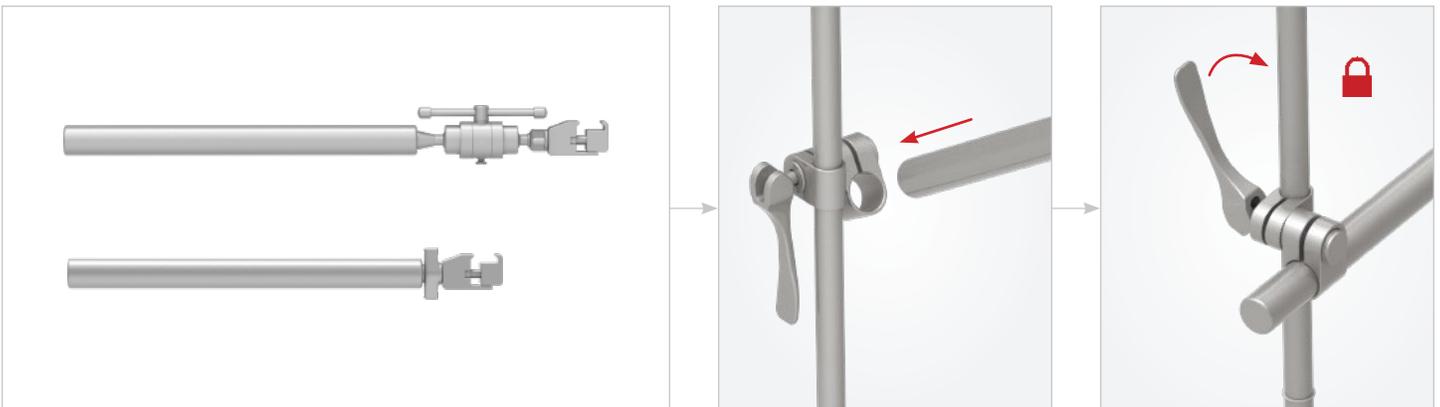

(OPTIONAL) STEP 2A: ARTICULATING HORIZONTAL BAR


If using Articulating Horizontal Bar, position as desired, ensuring ring clamp is horizontal and facing up. Turn sliding handle clockwise to tighten.

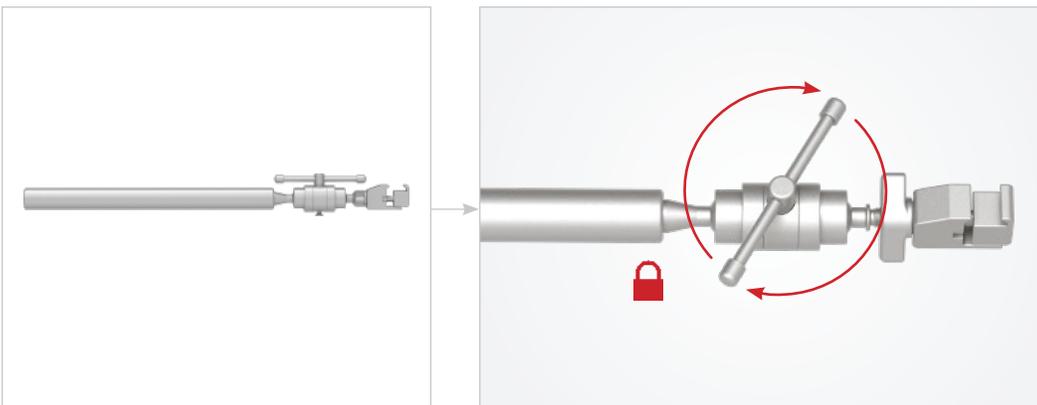

RingTrack™ Frame Set Up Instructions (Continued)



(OPTIONAL) STEP 3A: ASSEMBLE SEGMENTED RING

If using segmented ring, align the serrations on male and female ends. Insert, and turn knob clockwise to tighten.

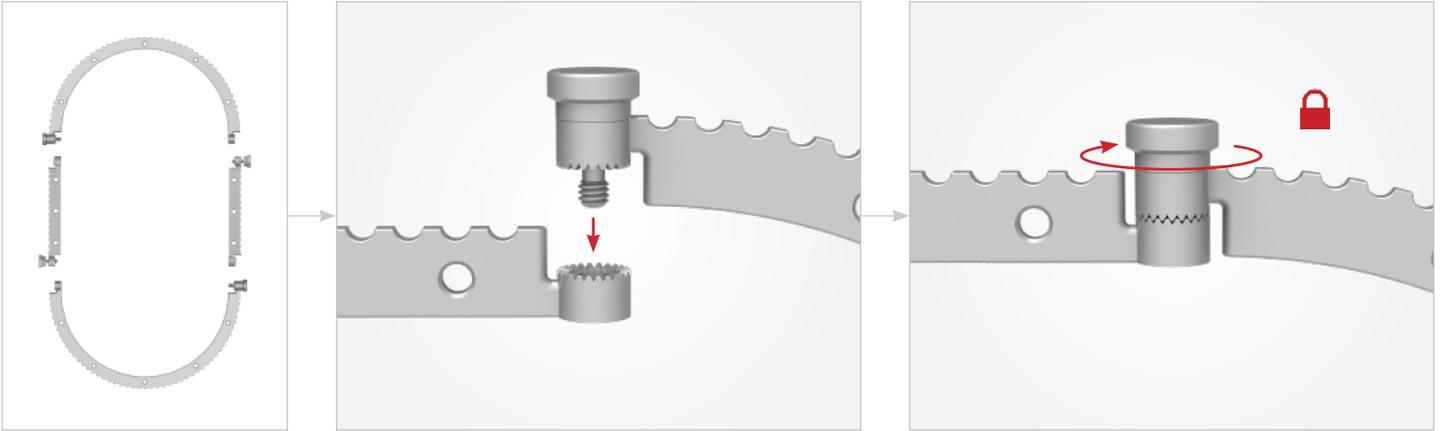

STEP 3: ATTACH RING


Select the appropriate ring for procedure. Place ring inside ring clamp on distal end of horizontal bar and turn knob to secure.

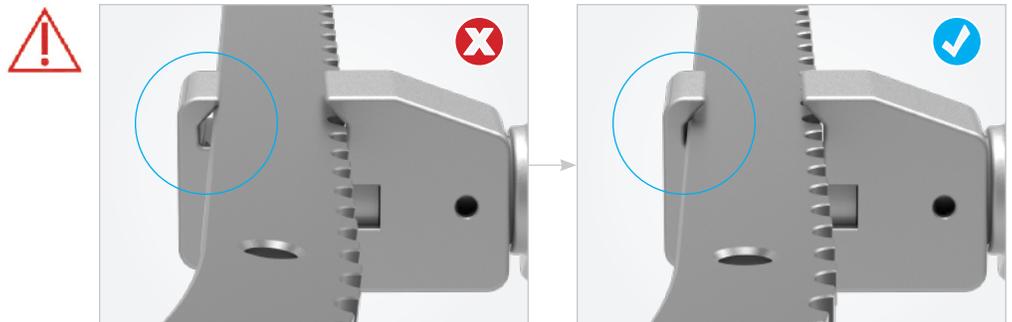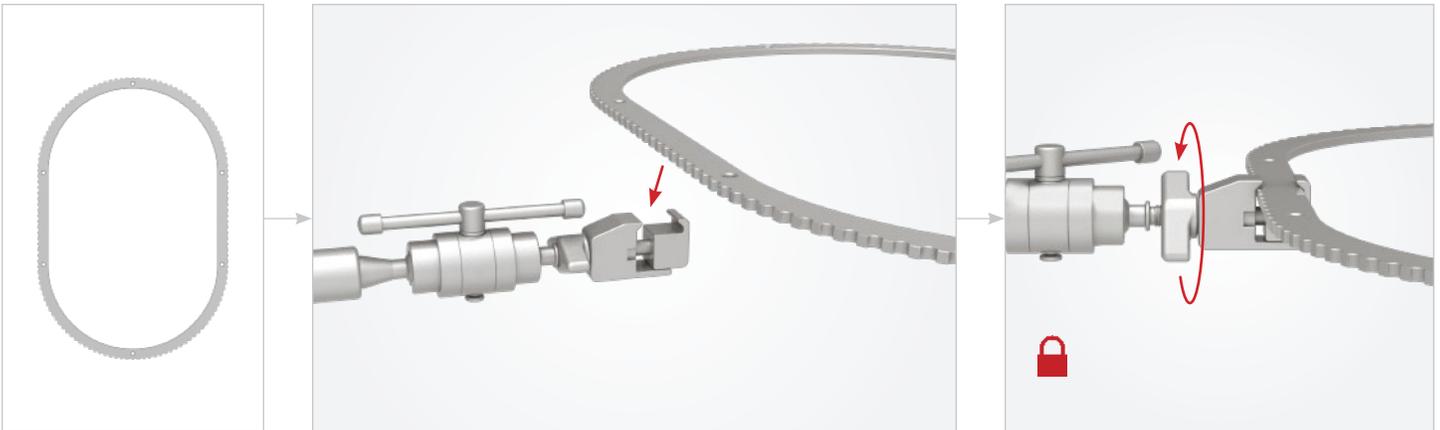

Ratchet and Handle Instructions


RATCHETS AND HANDLES

Insert blade handle into ratchet of choice and slide ratchet onto ring in desired location, retracting as needed.

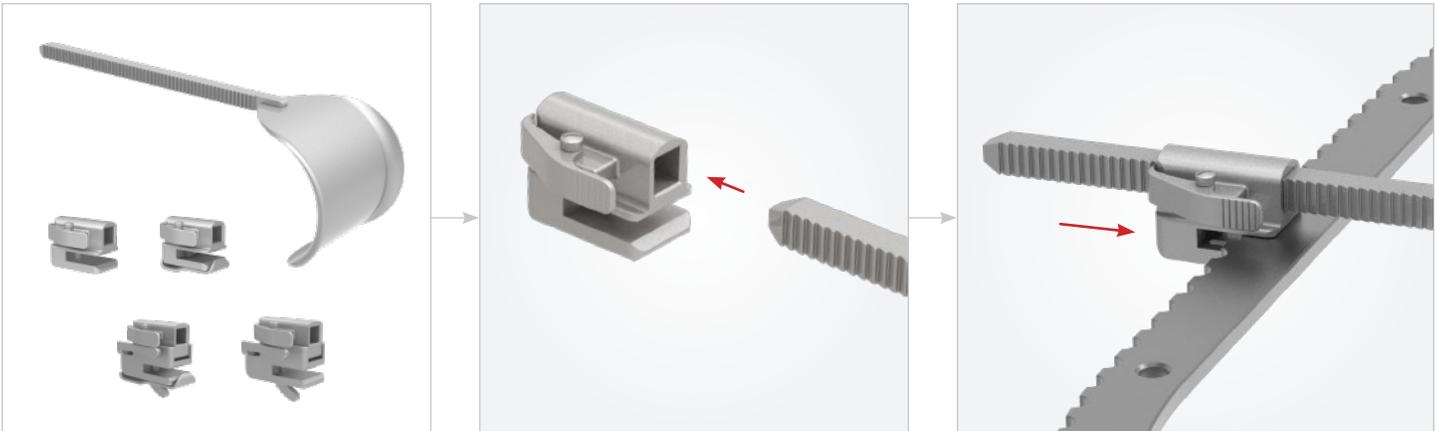


NOTE:

Alternately, ratchet may be slid onto ring before adding retractor.

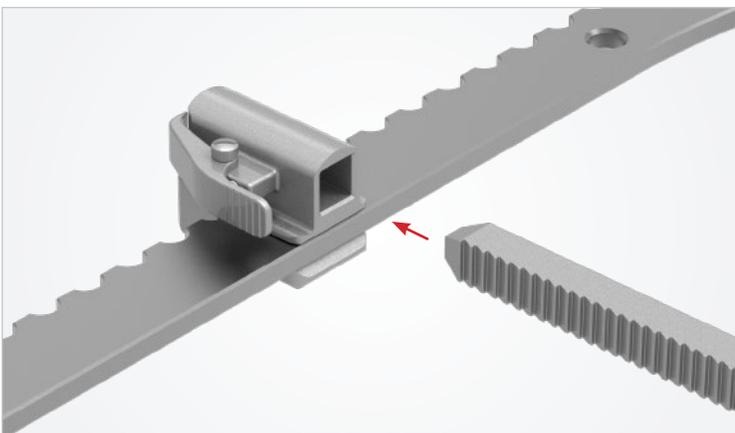

TO RELEASE RETRACTION


Press upper grooved pawl on ratchet.

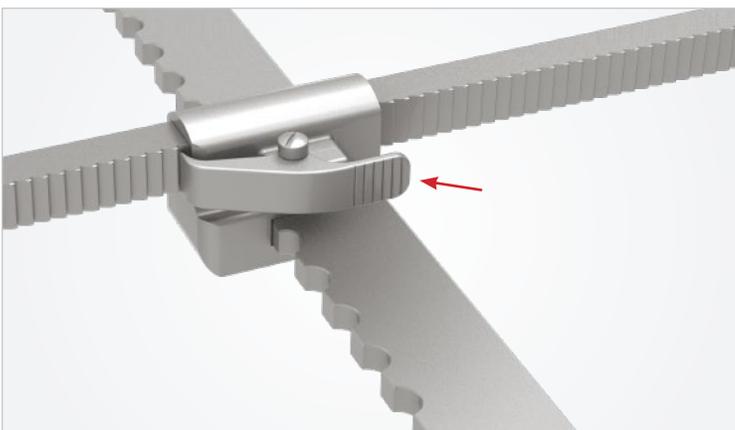

Optional Ratchet Instructions


(OPTIONAL) ANGLING RATCHETS

Angle blade in ratchet as desired. To release, press lower pawl on ratchet.

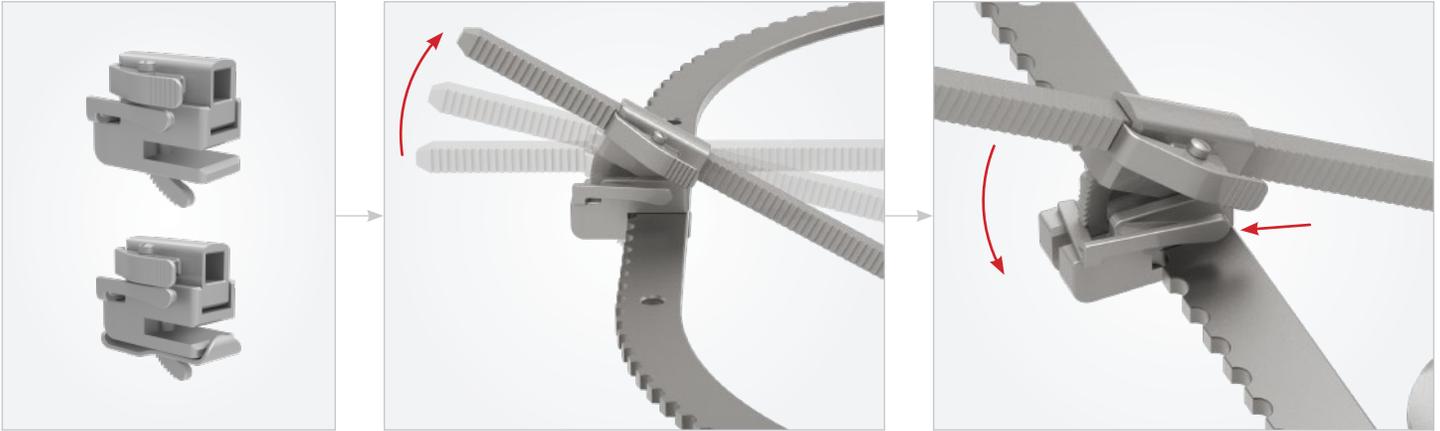

(OPTIONAL) CLIP RATCHETS


To release clip ratchet from frame, press back of clip and slide back.

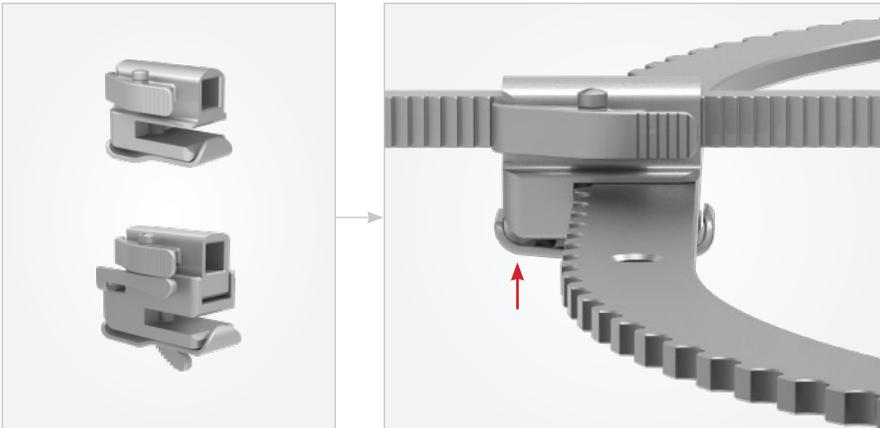

Optional Handles + Blades Instructions

(OPTIONAL) S-LOCK® HANDLES + BLADES

Attach blade to handle by pushing plunger and inserting blade nipple. Push plunger to release blade and switch between 'swivel' and 'locked' positions.

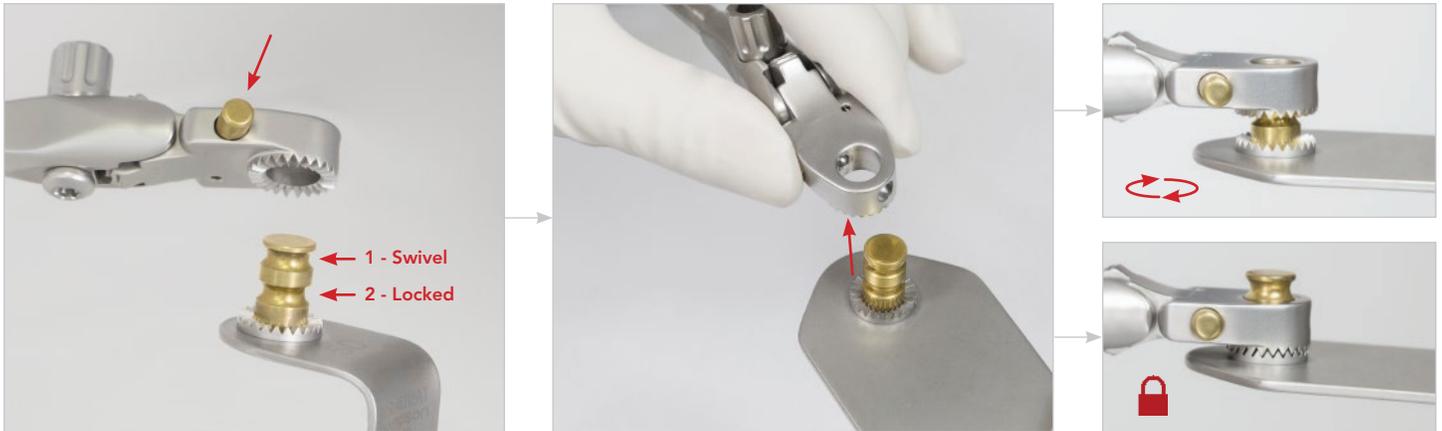

(OPTIONAL) QUICK ANGLE HANDLES

Blades may be angled 45° up or down by hand or with T-Handle.



For a Free Trial Call Today*

1.800.227.7543

Rev B
011123
dc rtum



Uncompromised Exposure

VISIT US ONLINE: THOMPSONSURGICAL.COM



Thompson
SURGICAL INSTRUMENTS

10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

EC REP

Emergo Europe

Westervoortsedijk 60
6827 AT Arnhem
THE NETHERLANDS

* Free trial valid for U.S. customers only. Customers outside U.S. please call +1-231-922-0177 for availability.

© 2020 Thompson Surgical Instruments, Inc. Printed in the U.S.A.

® S-Lock®, PLA®, and the "T Circle" logomark are Registered Trademarks of Thompson Surgical Instruments, Inc.

™ RingTrack™ is a Trademark of Thompson Surgical Instruments, Inc.

Patents: trpat.com

Other patents pending.



CE 0297

**Made In
USA**