(FR) FRANÇAIS

Infinite Height Rail Clamp

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Infinite Height Rail Clamp

Low Profile, Uncompromised Exposure

Thompson's Infinite Height Rail Clamp (IHRC) can be easily adjusted to any height between 13" - 22", allowing versatility for multiple procedures and patient sizes. The adjustable design prevents the post from interfering with operating space. The stable design can be used for both unilateral and bilateral set ups.

Choose an Infinite Height Rail Clamp with one or two cam joints to replace any standard Elite Rail Clamp for unilateral or bilateral configurations.

Step 1

Open rail clamp jaw and place on bedrail over the sterile drapes.

Step 2

Rotate the knob at the bottom of the rail clamp clockwise to tighten, utilizing the attached hanging handles for leverage when needed.

Step 3

Slide the post to the desired height and turn the handle clockwise to tighten.

NOTICE:

Do not attempt to adjust rail clamp height with frame components attached unless supporting the weight of the frame components with a hand. The post will not slide easily during height adjustments if this is not done.

ORDERING INFORMATION:

IHRC with 1 Cam III Joint (#43902IRC)

IHRC with 2 Cam III Joints (#43903IRC)

 **Thompson**
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

For a Free Trial Call **1-800-227-7543** *

* Free trial valid for U.S. customers only. Customers outside the U.S.A., please call +1-231-922-0177 for availability.

© 2018 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

® S-Lock®, PLA®, and the Thompson "T Circle" logomark  are Registered Trademarks of Thompson Surgical Instruments, Inc.

Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064
Other patents pending.

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
THE NETHERLANDS

CE 0297

Rev. C
051118
ihrcssum

**Made In
USA**