(PT) PORTUGUÊS

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Vprašanja ali dodatne informacije: +1-231-922-0177 / thompson surgical.com


Infinite Height Rail Clamp

Low Profile, Uncompromised Exposure

Thompson's Infinite Height Rail Clamp (IHRC) can be easily adjusted to any height between 13" - 22" , allowing versatility for multiple procedures and patient sizes. The adjustable design prevents the post from interfering with operating space. The stable design can be used for both unilateral and bilateral set ups.

Choose an Infinite Height Rail Clamp with one or two cam joints to replace any standard Elite Rail Clamp for unilateral or bilateral configurations.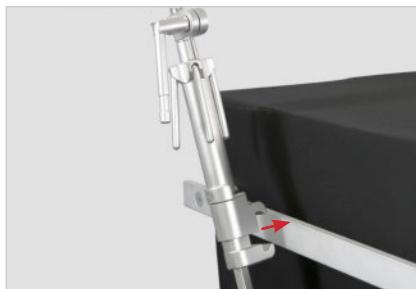

Step 1

Open rail clamp jaw and place on bedrail over the sterile drapes.

Step 2

Rotate the knob at the bottom of the rail clamp clockwise to tighten, utilizing the attached hanging handles for leverage when needed.

Step 3

Slide the post to the desired height and turn the handle clockwise to tighten.

NOTICE:

Do not attempt to adjust rail clamp height with frame components attached unless supporting the weight of the frame components with a hand. The post will not slide easily during height adjustments if this is not done.

ORDERING INFORMATION:

IHRC with 1 Cam III Joint (#43902IRC)
IHRC with 2 Cam III Joints (#43903IRC)